Protège seau
CareBag®

FICHE TECHNIQUE

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1- Produit

1.1- Description

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1- Product

1.1- Presentation

Commode pail liner with drawstring ties containing a super absorbent pad (Absorption capacity: 450 ml / 15 fl oz 0.9% saline solution equivalent) intended for the collection of excreta.

1.2- Technical information

Liner: Polyethylene, with blue drawstring. Useful dimensions: 52x38 cm / 20.5x15 in, Thickness: 37 µm.

Superabsorbent pad: Support based on cellulose and sodium polyacrylate. Dimensions: 24x13 cm / 9.4x5.1in.

2- Conditions of Use

2.1- Instructions for use

1. Open the box and take off the roll of commode liners.
2. Roll out the 1st commode liner and pull along the perforated mark line.
3. Open the liner, place it over the pail, refold the commode seat.
4. The absorbent pad must be at the bottom of the commode liner.
5. Use the commode with the commode liner as usually.
6. Wait for the content (+/-450 ml / 15 fl oz max) to turn into gel.
7. Close the liner with a knot.

2.2- Precautions for use

For single use, in case of reuse risk of insufficient or no absorption and therefore of potential contamination and discomfort for the patient. In case the pad comes into contact with the eye(s) or mucous membranes, rinse with plenty of water. In case of persistent discomfort, contact your doctor. Do not use on damaged skin. Keep away from children. In case the pad is swallowed, seek medical attention immediately and show the case.

2.3- Storage and Disposal

The product is sensitive to humidity and should be stored away from sunlight. Dispose of in household waste, or according to the excreta management protocol in force in the establishment.

3- Regulation

3.1- Conformity

CE Marking - Complies with the current European Regulation on medical devices.

3.2- Identification

Each product is identified with a lot number.

3.3- Customs Code

3923210000

3.4- Safety

This product is not considered hazardous according to EU regulations. Safety data sheet available on request.

4- Packing

CATALOGUE REF.	SALES UNIT	CASE PACKER	PALLET
7831738	Dispenser box content: Pack of 20 liners 200x115x115mm 0.6kg / 1.32lbs	18 packs of 20 liners (360 liners total) 415x365x365mm 14kg / 30.86lbs	30 cartons - 540 packs (10800 liners) 120x80x200cm 430kg / 947.99lbs (incl. pallet) (5 layers of 6 cartons)

- Dimensions: format L (length) x W (width) x H (height); sales unit/case packer: external dimensions
- Dimensions, weight, volume: ± 5%

CLEANIS S.A.S.U
22 avenue Aristide
Briand 94110 ARCUEIL

Web:
www.cleanis.eu/en