1- Produit

1.1- Description

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1- Product

1.1- Presentation

Disposable, double drawstring vomit bag containing a super absorbent pad. (Absorption capacity: 450 ml / 15 fl oz 0.9% saline solution equivalent) intended for the collection of vomiting.

1.2- Technical information

Bag: Polyethylene, double drawstring (blue & white). Useful dimensions: 25x25 cm / 9.8x9.8 in, Thickness: 32 µm.

Superabsorbent pad: Support based on cellulose and sodium polyacrylate. Dimensions: 24x13 cm / 9.4x5.1 in.

2- Conditions of Use

2.1- Instructions for use

1. Open the box and take one device.
2. Unfold the device, open it and hold it with both hands.
3. Vomit in the bag.
4. Wait for the content (+/- 450 ml / 15 fl oz max) to turn into gel.
5. To close the device, pull strings on both sides of the device at the same time, the blue one on one side, the white one on the other side. Tie up.

2.2- Precautions for use

For single use, in case of reuse risk of insufficient or no absorption and therefore of potential contamination and discomfort for the patient. In case the pad comes into contact with the eye(s) or mucous membranes, rinse with plenty of water. In case of persistent discomfort, contact your doctor. Do not use on damaged skin. Keep away from children. In case the pad is swallowed, seek medical attention immediately and show the case.

2.3- Storage and Disposal

The product is sensitive to humidity and should be stored away from sunlight. Dispose of in household waste, or according to the excreta management protocol in force in the establishment.

3- Regulation

3.1- Conformity

CE Marking - Complies with the current European Regulation on medical devices.

3.2- Identification

Each product is identified with a lot number.

3.3- Customs Code

3923210000

3.4- Safety

This product is not considered hazardous according to EU regulations. Safety data sheet available on request.

4- Packing

CATALOGUE REF.	SALES UNIT	CASE PACKER	PALLET
7733987	Dispenser box content: Pack of 20 devices 250x50x150 mm 0.5 kg / 1.10 lbs	21 packs of 20 devices (420 bags total) 770x380x170 mm 11kg / 24.25 lbs	33 cartons - 693 packs (13860 devices) 120x80x205 cm 355 kg / 782.64 lbs (incl. pallet) 11 layers of 3 cartons

• Dimensions: format L (length) x W (width) x H (height); sales unit/case packer: external dimensions
• Dimensions, weight, volume: ± 5%

CLEANIS S.A.S.U
22 avenue Aristide
Briand 94110 ARCUEIL

Web:
www.cleanis.eu/en